



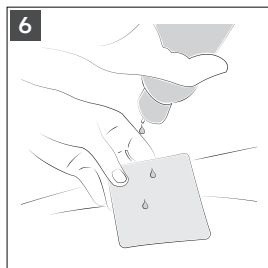
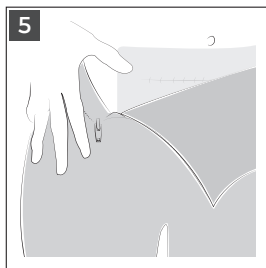
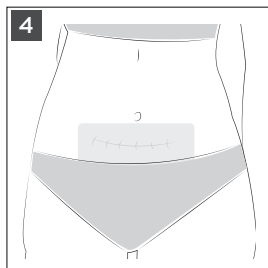
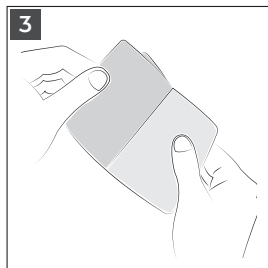
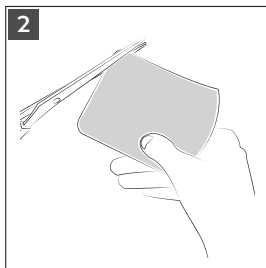
amoena

RECOVERY CARE / AMOCURA CURASCAR SILICONES / SILIKONNARBENPFLASTER


Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsanvisning. Instruksjoner for bruk. Användningsinstruktioner. Instrukcja użycia. Käyttöohjeet. Návod k použití. Инструкции по применению. Οδηγίες χρήσης. 使用説明. 使用説明書. مداخلت سالانہ ایملعت

Deutsch	DE	5
Englisch	ENG.	10
Français	FR	15
Español	ES	20
Italiano	IT.	25
Nederlands	NL	30
Dansk	DK.	35
Norsk	NO.	40
Svenska	SE	45
Polski	PL	50
Suomalainen	FI.	55
Český	CZ	60
Русский	RU	65
Ελληνικά	GR	70
中國	CN	75
日本の	JP	79
84	ARAB العربية	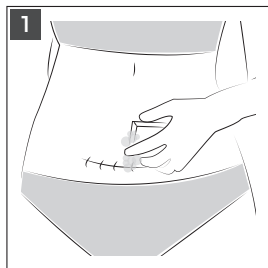

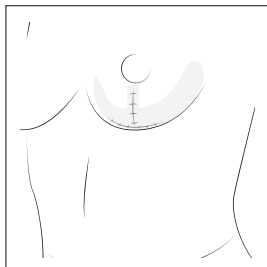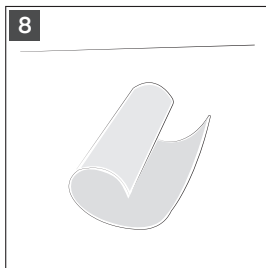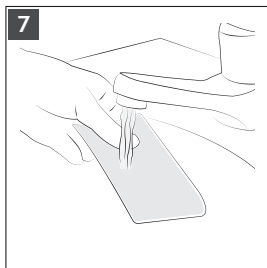Anchor

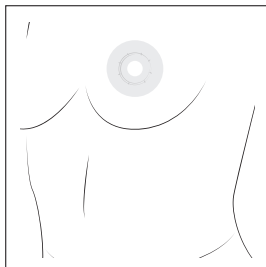Mammilla Circle

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Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. For more than 45 years, Amoena has been synonymous with innovative and high-quality products that support the needs of women following breast surgery.

Intended purpose

In scar therapy, silicone and compression provide the ideal combination. In compression treatment, a therapeutically effective pressure is also exerted on the closed wound area. In this way, excessive connective tissue remodelling can be counteracted.

The combination of constant surface pressure from the compression garment, along with silicone, has a positive influence on scar healing. The silicone creates a moist environment on the scar, which prevents the scar from drying out. It keeps the scar soft and has a positive effect on scar tissue.

Indication

- Prevention and treatment of impaired scars
- Treatment of red and raised scars
- After surgical procedures (e.g. after reconstructive or aesthetic surgery)
- Treatment recommended in combination with compression garments

Contraindications

- Do not use on open wounds
- Do not use with skin irritation
- Do not use with known intolerance to the ingredient.
- Should skin intolerance occur during use, please do not use the silicone scar patch anymore.

Side effects

No side effects are known in the event of proper use. However, should negative changes, such as skin irritation occur during prescribed use, please contact your doctor or medical retailer immediately. If there is known intolerance to one or more ingredients of this product, please consult with your doctor before use. If your symptoms worsen during application, please consult your doctor immediately. The manufacturer is not liable for damages/injuries that have arisen due to improper handling or non-intended use.

General care instructions

- Clean the silicone scar patch with a few drops of the Amoena Soft Cleanser or a non-greasy soap (e.g. mild dishwashing liquid) and wash thoroughly by hand once every day (Fig. 6).
- Rinse the silicone scar patch under warm running water (Fig. 7).
- The silicone scar patch can be rolled up while still wet (Fig 8). The adhesive strength disappears completely.
- To dry, place the rolled-up silicone scar patch on a smooth surface, with the adhesive side facing up until the water has evaporated from the surface.
- The silicone scar patch is adhesive again, once it is completely dry.
- The protective film can now be applied again.
- Finally, apply and use the silicone scar patch while it is dry.
- When the silicone scar patch is not being worn, it should be stored in a clean, dust-free space after washing and drying, with a protective foil on the adhesive side.

In the event of non-compliance with the hygiene instructions, skin irritation may occur due to germ growth underneath the silicone scar patch; the manufacturer may not be made liable for this.

PRODUCT-SPECIFIC NOTES

Silicone scar plaster

Art. No.	Article	Dimensions
010	Stripes	10.24 x 1.38 inch
011	Squares	10.24 x 3.94 inch
012	Mammilla Circle	Ø outside 2.95 inch, Ø inside 0.79 inch
013	Anchor	9.06 x 5.71 inch

All silicone scar patches bear the colour designation 'clear'.

Instructions for use

- First clean the scar and surrounding skin thoroughly. Keep the scar tissue dry and free of grease. (Fig. 1)
- If necessary, you may cut the silicone scar patch into the desired shape and size. Make sure to cut shapes that are as round as possible, without corners. (Fig. 2).
- Remove the protective film from the silicone scar patch (Fig. 3).
- Carefully apply the self-adhesive side of the silicone scar patch onto the scar (Fig. 4).
- For an optimum result, the scar must be covered completely with the silicone scar patch. Ideally, this overhangs the scar by at least 1 – 2 cm.
- Now you can carefully pull your compression garment over the silicone scar patch (Fig. 5).
- When removing, please ensure that the silicone scar patch is pulled off from different areas.
- Then use a moisturising skin cream.
- With proper application, the silicone scar patches can be used for up to 12 weeks.

Material composition

Stripes	Polydimethylsiloxane, polyurethane film, polyethylene protective film, polyethylene cover strips
Squares	Polydimethylsiloxane, polyurethane film, polyethylene protective film, polyethylene cover strips
Mammilla Circle	Polydimethylsiloxane, polyurethane film, polyethylene protective film, polyethylene cover strips
Anchor	Polydimethylsiloxane, polyurethane film, polyethylene protective film, polyethylene cover strips

Storage instructions and shelf life

Please enquire about regional disposal regulations.

Special instructions

Store in a dry place

Protect from sunlight

Single patient – multiple use

Disposal instructions

Please enquire about regional disposal regulations.

Reporting obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death.

Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

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Service information

Do you have any questions?

If so, we would like to help – please call us or write to us.

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This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are approximately 20 lines visible. The paper has a slight shadow on its right side, suggesting it's resting on a surface.

[illegible]

Australia

Amoena Australia Pty.Ltd.
107 Fergie Street
Fitzroy North, VIC, 3068.
Australia
www.amoena.com.au

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Amoena Italia srl
Via Modigliani 25/27
20090, Segrate (MI)
www.amoena.it

Norway

Amoena Norge AS
Ullernsschausséen 64–66
www.amoena.no

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Amoena Sweden AB
Olof Palmes gata 14
111 37 Stockholm, Sverige
www.amoena.se

United Kingdom

Amoena UK Ltd
1 Eagle Close,
Chandlers Ford,
Eastleigh, Hampshire,
SO53 4NF
www.amoena.co.uk

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2020 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH, 83064 Raubling, Germany